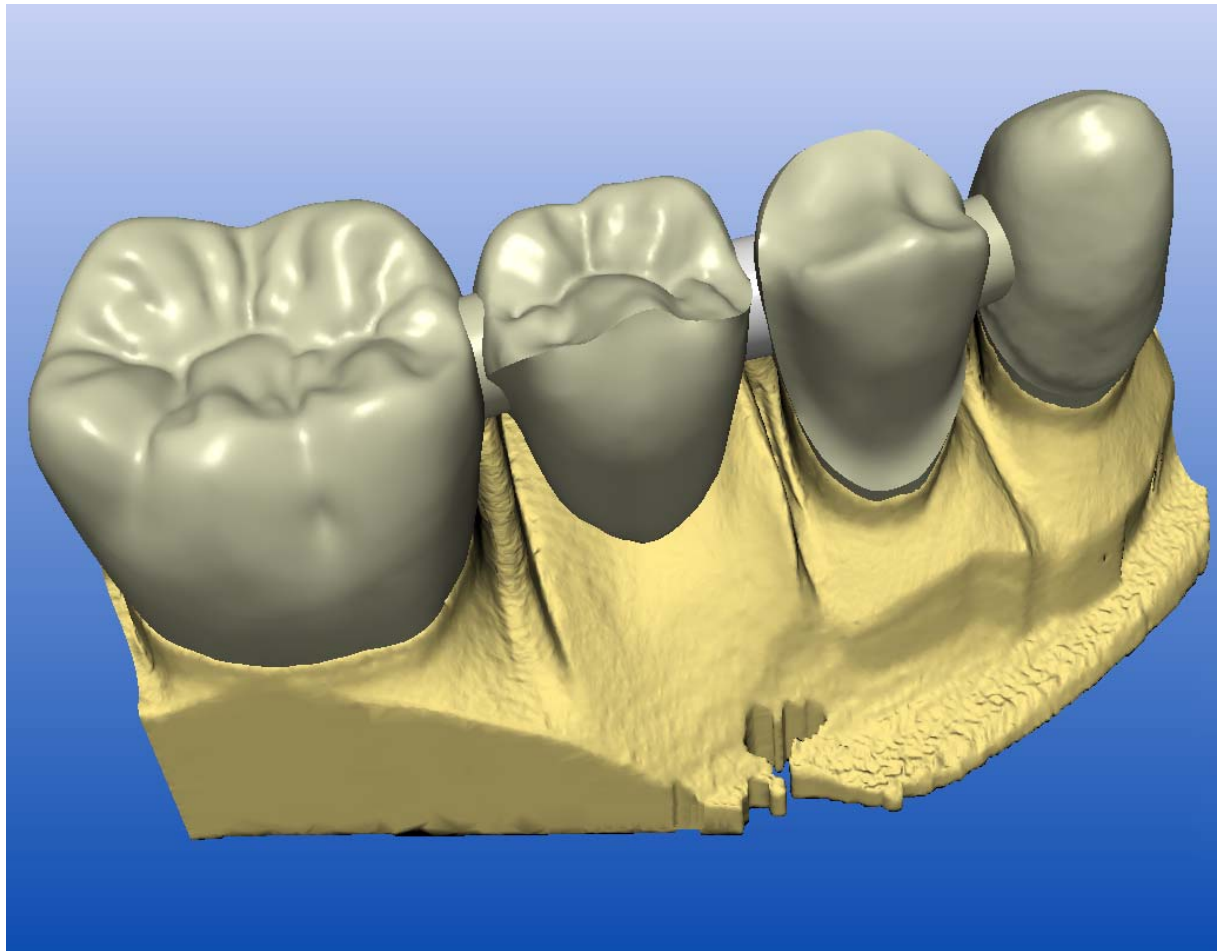
# inLab 3D

Service Pack V3.04

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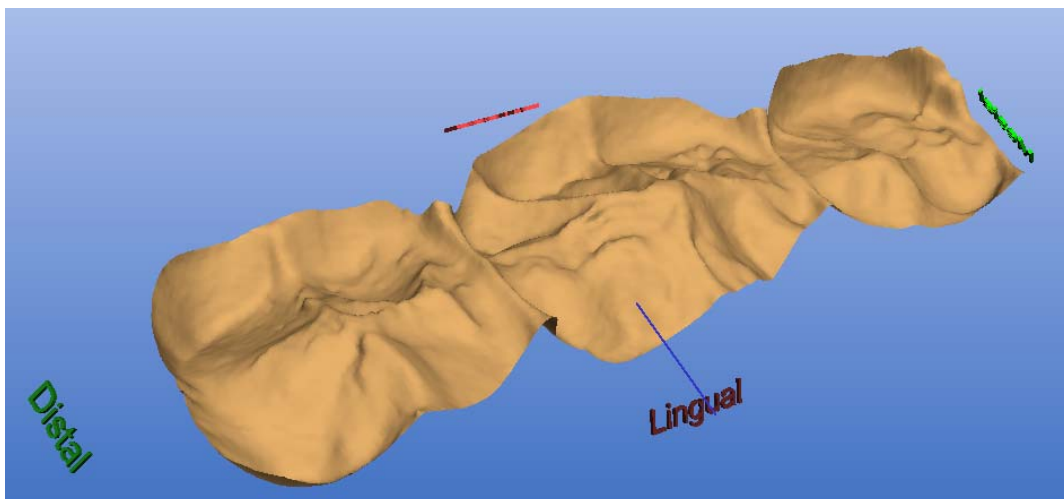
*Beschneiden des Antagonisten*

## Installation

The inLab Service Pack V3.04 adds several useful functions to the inLab 3D software. It requires prior installation of inLab 3D software V3.00, V3.01 or V3.03. To install it, follow the instructions displayed on screen. Following successful installation, software version V3.04 is displayed.

*inLab 3D V3.04 Welcome dialog*

## Large blocks

You will find a new large block under Sirona in the block selection menu for inLab MC XL (and CEREC MC XL).

inCoris ZI 65/25

You will find a new large block there for VITA:

In-Ceram YZ 65/25

You will find new large blocks there for Ivoclar Vivadent:

IPS e.max ZirCAD B55

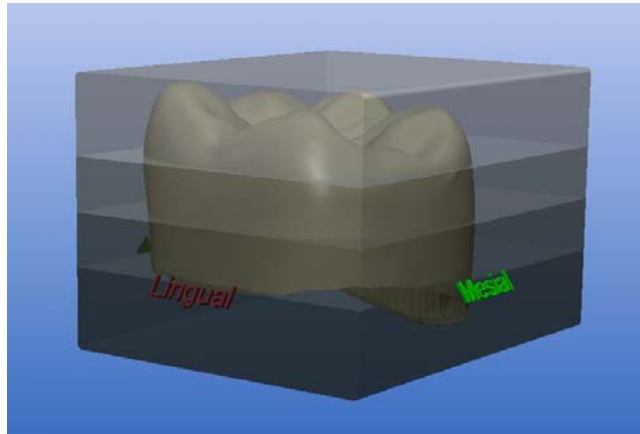
IPS e.max ZirCAD B65

Sirona inCoris ZI 65/25

## VITA TriLuxe Forte

It is now possible to select the polychromatic view for VITA Triluxe Forte in the milling preview. The four layers of the block are then displayed on the screen. TriLuxe Forte appears as a separate material in the block selection menu.

*VITA TriLuxe Forte preview*

## Ivoclar Vivadent IPS Contrast Spray Labside

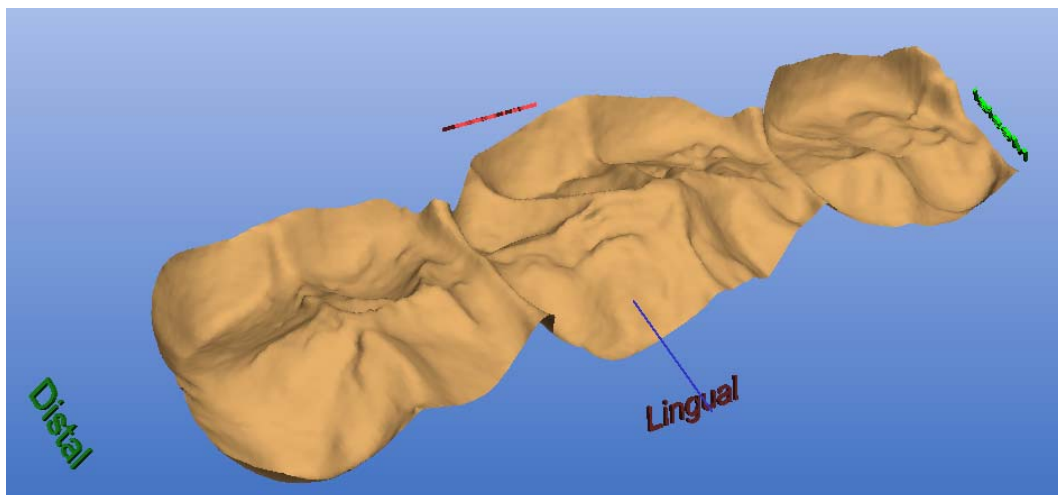
In the model materials selection dialog box for optical impressions with inEos you will find

Ivoclar IPS Contrast Spray Labside

as the new entry.

## Trimming the antagonist as a mandatory step with inLab

Following reconstruction of the 3D models from the scan data and trimming of the preparation, the software automatically prompts you to correctly trim an additionally scanned antagonist as well. This new design step will automatically be skipped if no bite registration has been scanned.

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**Sirona Dental Systems GmbH**
Sirona Dental Systems LLC